

Instructions for use:

NaCl by PREVOR, Emergency ocular washing solution

1. WHEN TO USE NaCl BY PREVOR

NaCl by PREVOR is an emergency ocular dust rinsing solution **without chemical hazard.**

2. HOW TO USE NaCl BY PREVOR

Look down. Apply the eye washer on the eye. Moove your head back. When the solutions recover the eyelid, open your eyes and flow out slightly the solution to create a mecanical flow in order to bring the dust. The Efficiency of NaCl by PREVOR is optimize *est optimisée lorsqu'il y a un léger écoulement pour créer un flux mécanique.*

Wash until dust departure and think to wash the conjunctival sac.
If discomfort persists, contact a medical specialist

The NaCl by PREVOR, a neutral PH, is an ophthalmic sterile solution. It can be use even in case of damaged cornea.

According to the EN15154 standard, the ideal temperature at which it should be used lies between 15-35°C.

3. WARNING

NEVER DELAY THE WASHING IN CASE OF CHEMICAL ACCIDENT.

As all the solution on the basis of sodium chloride, even a neutral PH, The NaCl by PREVOR action is limited by a mechanical drive, and it is insufficient when rising chemical products (corrosive and irritating). In that case, use, in first intention, Diphotérine® or Hexafluorine® solutions, a specialist should be consulted.

If Diphotérine® or Hexafluorine® sotions are not available, the washing can be start with the NaCl by PREVOR, even like washing with water, this one should be extend during at least 15 minutes, and so, without negligible benefit.

In case of persistent trouble or the presence of a foreign object after washing, a specialist should be consult.

Single use only. Do not reuse, risk of cross-contamination.

Contraindication

No contraindication currently known.

Use Precautions

Do not exceed the expiry date specify on the flask.

4. ADVERSE EFFECTS

None

5. CONSERVATION

It is also advised not to expose the product to freezing temperature, because the aqueous solution can freeze and thus may not be immediately usable. However, there is no loss of effectiveness when the solution has thawed out.

Do not use in case of packaging, flask or eye washer deterioration.

6. ADDITIONAL INFORMATION

The NaCl by PREVOR is a Iia class medical device. The solution is a non-irritating, non-allergenic and non-toxic solution.

Laboratoire PREVOR Moulin de Verville
www.prevor.com 95760 Valmondois
 +33 1.30.34.76.76 FRANCE



PREVOR
 PRÉVOIR ET SAUVER
 Laboratoire de Toxicologie & Maîtrise du Risque Chimique

CE 0459

